IDENTIFICATION SHEET

Final inspection:

Signature:

Identification tag / Product type (write up):

EN

DE

Warranty conditions:

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The warranty has to be applied in the business organization, where the product was bought, or in authorized service repair shop.
3. **Properly filled Certificate of Warranty and a copy of acquiring document have to be advanced when applying the warranty. Do not pull down the identification card. It has to be readable and undamaged for the whole warranty period.**
4. Any defect that has occurred must be repaired immediately after finding it, the user is fully responsible for defects resulting from the continued use of the product without the original defect being repaired.
5. The product has to be used entirely to the purpose, which it is determined for, the user has to follow the instructions mentioned in the instructions for use. Producer / distributor is not responsible for defects caused by incompetent handling in conflict with the instructions for use.
6. The product has to be used, treated, maintained and stored properly according to directions mentioned in the instructions for use.
7. The warranty proceedings can be initiated, if the product under claim has been sent complete in original package, carefully packed and secured against harm. The owner is obliged to advance the product under claim properly cleaned up, hygienically flawless due to common hygienic principles. The owner cannot ask the seller to do basic maintenance or cleaning of the product during claim proceedings.
8. Every defect has to be claimed immediately
9. The warranty is forfeit if the defects were caused by forbidden overloading, accident, violence or incompetent intervention into the product or incompetent handling.
10. Warranty period cannot be interchanged with using period, i.e. period, for which, when used and treated properly including maintenance, the product, in respect of its characteristics, given purpose and difference in construction, can be used properly.
11. The engineering solution does not guarantee the ideal direction trails.

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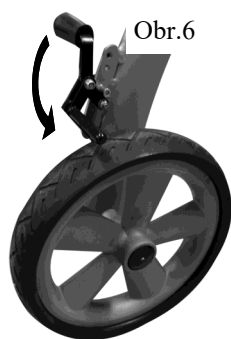
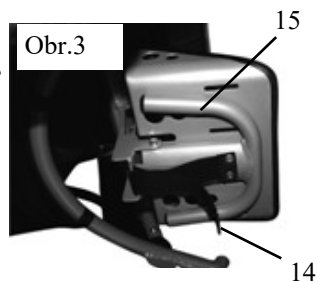
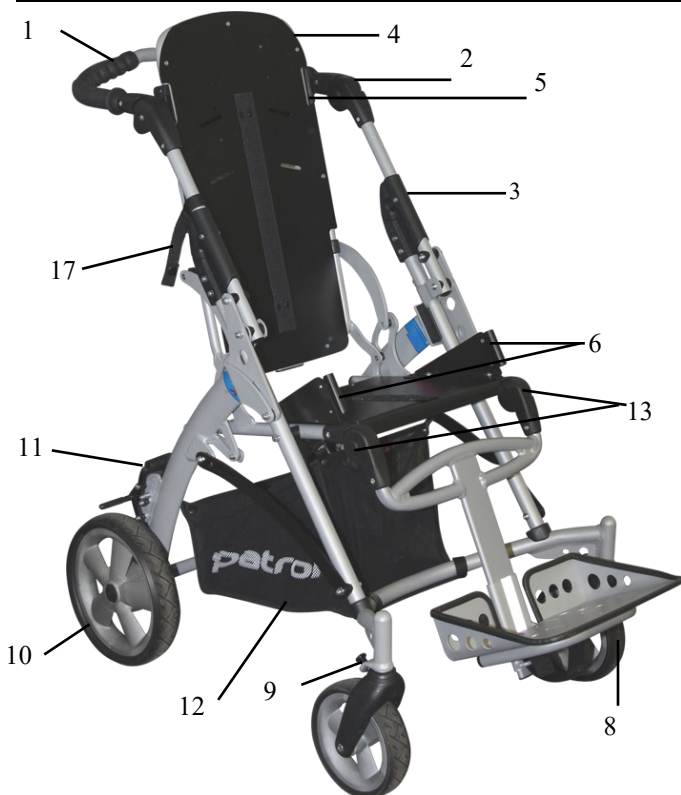
Dear customers,

JACKO 5 is light and compact rehab-stroller of a new generation and is designed for short as well as for long stay of handicapped children. The basis of the stroller is chassis made from light aluminium alloys, guaranteeing low weight and easy manoeuvrability.
In case you are not fully sure about the proper usage of the product, feel free to contact your local dealer or service centre.

SAFETY AND WARNING NOTES:

① **CAUTION! Any load attached to the handle (bag, ...) affects the stability of the stroller!**

② **The side padded frame covers must be used properly all the time!**

③ **Recommended tire air-pressure: 230 / 230 kPa**

⑥ **Footrest velcro lock must be properly tightened while using stroller!**

maximum load.

weight of the product

- **WARNING! Keep fingers out of any moving parts when operating with the stroller.**
- **WARNING! Never leave your child unattended! Even if the belts and brakes are in use.**
- **WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.**
- **WARNING! Check before use that seat unit is properly locked in the chassis interface.**
- **WARNING! The seat unit is not suitable for a child younger 6 months.**
- **WARNING! This product must not be used while jogging or in-line skating.**
- **WARNING! The seat unit must not be used in motor vehicles as the car seat.**
- **ATTENTION! Read this manual carefully before the first use of the stroller, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.**

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc, are tucked in, so they do not fall in the wheels.
- The stroller **MUST NOT** be used as a seat for a child while traveling in motor vehicle or bus.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- Change of the seat direction must be done without the occupant and with empty seat unit.
- Do not allow the child to climb unassisted into and out from the stroller.
- Always apply the parking brakes, when getting in and out the stroller.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.
- Do not stand on the footrest.
- The brakes must be engaged at all times when the stroller is not moving.
- Getting over the curb practice firstly with an empty stroller.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the stroller.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the stroller. Overloading of a chassis, seat, bag or basket causes reduction of stability of the stroller.
- Make the full attention, when manipulate with the stroller, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.
- Always lift the stroller in two persons.
- The stroller is not designed to be driven down the stairs.
- Never transport the stroller when you and/or assistance are not sure if you can transport the stroller with the child safely.
- When you drive the stroller on the road, follow the road traffic regulations.
- Do not use the stroller for any other purposes then it is designed for.
- The stroller is designed to carry only one child.
- Do not use any other than original PATRON accessories designed for this product.

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SCHEMA - POPIS:

- When the stroller is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.
- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.
- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.

INTRUCTIONS FOR USE

1) Chassis:

Assembling the stroller (with attached seat):

Place folded stroller on wheels. Unbutton and release the transport safety-catch on frame(17). To open the undercarriage stay between the rear wheels and hold handle tightly. Move handle up and backwards until the handle folding release is fully locked in its end position (3). To check the locking position, move upwards the handle to ensure that both mechanism are fully locked (the undercarriage must not start folding). Pull the padded frame covers downwards over the locking mechanism and close both covers by zippers properly .Release backrest recline adjustment lever (18) and set the position of backrest. Tighten lever properly. Release the footrest angle adjustment lever, set the position of footrest and tighten lever again. Unfold the footplate and lock it by velcro fastener. Attach the safety bar. Attach and adjust fixation accessories if needed. The stroller is assembled.

Folding the stroller (with attached seat):

Apply the parking brake and take off the safety bar. Press both buttons on footrest hinge simultaneously and set the position of footrest a bit less than horizontally ca 20 degrees. Release the footrest velcro lock and close the footplate. Release the backrest recline adjustment lever and push backrest as far as possible forward, towards the seat base. (It is recommended to take out the exceeding fixing accessories and adjust the seat width by side plates to the maximum limit, which enables to decrease the size of the folded seat). Tighten the lever. Unzip the padded frame covers partially and pull over the locking mechanism upwards. Lift the handle folding release movable parts on both sides and fold the handle forward towards the front wheels. Ensure folded undercarriage by transport safety-catch. The stroller is folded

Parking Brake:

To engage the brake, press brake lever pad on both sides fully till the end position. To disengage the parking brake, lift both brake lever pads.

Swivel lock:

If needed, the front wheel forks can be locked against vertical rotation. To lock the castor forks lift and turn the swivel lock pin by 90° on both sides and let it to snap in in the designated hole on the castor fork.

Handle height adjustment:

To change the handle height position press the black plastic buttons on the handle hinges on both sides simultaneously (2), set the handle to desired position, release buttons. Handle will lock in new position automatically, you will hear the locking sound.

Safety Bar:

To take off the safety bar press pin release on both sides simultaneously and pull the bar upwards to remove it. To insert the safety bar, both pin releases have to be pointed ahead. Press slightly both press pins and insert the safety bar into the tube inlays at once until the pins are locked. Check the correct position of safety bar by slight pulling up, safety bar must not lift.

Backrest recline positioning:

Hold the seat backrest firmly. Loose backrest recline adjustment lever (18). Set the position of backrest. Tighten the lever properly. If needed unzip partially the zipper placed on the rear part of backrest cover to make the manipulation easier. Check the correct locking of the backrest position. Backrest must stay in preset position.

Footrest set angle positioning:

Press both buttons on footrest hinge simultaneously (13), set the angle position of footrest and release the buttons. Footrest will be locked in the new position automatically, you will hear the locking sound.

Footrest-set height positioning:

Loosen two hexagon-bolts placed on rear side of footrest slider profile. Set desired position of footrest. Properly tighten both hexagon-bolts.

Footplate angle adjustment:

Loosen two hexagon-bolts placed under the footplates. Set the angle of footplate by pulling or pushing of footplate angle adjustment slide-plate, which is determine requested angle of the footplate. Properly tighten both hexagon-bolts.

Footplate velcro lock:

Tighten and fasten at any time the velcro lock fastener placed under the footplate to ensure fixed footrest position. (security device against folding of footplate).

Standard equipment

Basket below the stroller :

Attaching – Attach the baskets centre part over the undercarriage middle cross bar using fasteners. Fix it with the poppers. Attach the baskets rear part, interlace the fasteners through the side loops placed on the rear frame profiles. Fix it with the poppers. Attach the baskets front part over the undercarriage front curved cross bar using fasteners. Fix it with the poppers .

Detaching – unfasten and release all fasteners from the anchoring points.

Transport safety catch:

Attaching – pull the transport safety catch fastener around the connecting bar and front frame vertical tube. Interlace the fastener through the loop and fix it with the popper.

Detaching – unfasten and release the fastener.

Optional equipment:

This product can be optionally supplied with very wide range of fixation and positional accessory. Please find the optional accessory user guide as separate brochure.

2) Stroller in operation:

Getting into the stroller:

Engage parking brake. If necessary ask trustful person for help to hold the stroller firmly. Place the child into the stroller carefully. Attach the safety bar and adjust the footrest position. Pull on and adjust all fixation and positioning accessories the stroller is equipped with. Always make sure that the child is seated comfortable in the stroller.

Getting out of the stroller:

Engage parking brake. If necessary ask trustful person for help to hold the stroller firmly. If required to easy getting out of the stroller adjust footrest position Take off all applied fixation and positioning accessories. Take off the safety bar and get the child carefully out of the stroller.

Getting over the curb stones and low obstacles:

Down - Drive the front wheels to the edge of the curb. Hold the handles firmly and tip the stroller backwards and balance on the rear wheels. Drive the rear wheels to the edge of the curb, lower the stroller down the curb slowly on its rear wheels.

Up - Drive the front wheels close to the curb. NEVER bump the front wheels into the curb stone!. Hold the handles firmly and tip the stroller backwards and balance on the rear wheels. Drive forward and place the front wheels on top of the kerb. Position the rear wheels at the curb tone bottom edge and lift to the upper level.

Manual transport of the stroller:

- make use of ramps and lifts when they are available

- if there is no alternative and steps should not be avoided at least one trustful assisting person must be available to give assistance

- do not attempt to take the stroller up or down over the number of steps unless you and your assisting person are not sure to have the full control during the manipulation as well as he or she can absorb the weight of its occupant and stroller.

- take the rests in regular intervals, if you are tired

- occupant hands must be inside the seat unit during the manual transport time

- NEVER lift the stroller holding the safety bar or footrest!

Going stairs downwards:

Tightly hold the handle and position the stroller squarely with the front wheels on the top step. Tip the handle backward and push the stroller forward until the rear wheels are at the end of top step. Assisting person stands in front, holding the stroller by gripping the front frame. Lift the stroller by both persons at once. Carry the stroller down the steps. Continue step by step to the bottom surface.

Going stairs upwards:

Tightly hold the handle and position the stroller squarely with the rear wheels against the bottom step. Tip the handle backward and push the stroller forward until the rear wheels are at the end of bottom step. Assisting person stands in front of stroller, holding the stroller by gripping the front frame. Lift the stroller by both persons at once. Carry the stroller upstairs. Continue step by step to the top surface.

Upholstery:

Seat upholstery is fully detachable and hand washable by lower temperature (max. 40°C).

Unzip the zipper of the upholstery on the back side of backrest. Pull the upholstery over the top of the backrest and attach to the backrest and seat by Velcro fasteners. Pull the upholstery over the seat side panels one by one. Fasten the zipper of backrest upholstery.

MAINTENANCE

The stroller has been designed to the high standard and with correct usage and maintenance will provide you many years of trouble free service.

Regularly inspect handle folding release, safety bar fittings, locking devices, catches, footrest, joints fixings and handle release safety clip to ensure they are secure, clean and in full working order. Brakes, wheels and upholstery of the stroller are expendable parts and should be replaced if necessary by a service centre.

It is recommended that the stroller is checked by an approved service centre every 12 months.

If the stroller is damaged or in need of any repair, only authorized service centres are allowed to provide this services.

Only spare parts supplied by company PATRON has to be used for repairs. Rotating and moving parts of the stroller treat regularly by spray lubricants (e. g. WD 40), minimally every six months. Never use oil or grease.

Ensure that the lubricant penetrates between the components but do not overspray as this may encourage the accumulation of dirt. Wipe off any spray excess.

Folding and assembling the stroller should be always easy. If not, please read again the appropriate parts of user guide.

Check regularly the tire pressure.

Wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.

The upholstery of the stroller can be washed by a cloth wetted in warm water or hand washed (max. 30°C)

Never clean with abrasive, ammonia based, bleaches and alcohol based cleaning compounds.

After exposure to damp conditions, dry the product off with a soft cloth and leave fully open in warm and dry environment.

Store your stroller in a dry, safe place with recommended temperature 5-20° C.

Check and tighten the screw connections regularly.

– We recommend that after 24 months visit a service department.

If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

NOTES

Changes on all the products of the company PATRON Bohemia a.s. are reserved.

SCHEMA DESCRIPTION

1.	Handle	12.	Basket
2.	Handle height adjustment hinge	13.	Footplate angle adjustment hexagon-bolts
3.	Handle folding release	14.	Footplate Velcro lock
4.	Backrest	15.	Footplate
5.	Hood bracket	16.	Padded frame cover
6.	Safety bar brackets	17.	Transport safety catch
7.	Footrest		
8.	Front swivel castor wheel		
9.	Swivel lock		
10.	Rear wheel		
11.	Parking brake		

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- Alternativ steht Ihnen PATRON Deutschland GmbH für Produktinformationen zur Verfügung.
- Es sollen nur Ersatzteile PATRON Deutschland GmbH verwendet werden.
Technische Änderungen an den Produkten der PATRON Deutschland GmbH sind vorbehalten.